



DATE	REV.	REVISION RECORD	ECN #	DR. CHK.
10-18-04	A	RELEASE & ISSUE	E04-121	LEG
4-18-07	B	REVISED AND REDRAWN	E07-003	LEG
9-12-07	C	ADDED 'J' SIZE CYLINDER	E07-098	LEG

COATED 45° BENT HANDLE W/COATED STAINLESS STEEL RING, AND FLANGE WASHER W/BOLT-ADAPTER

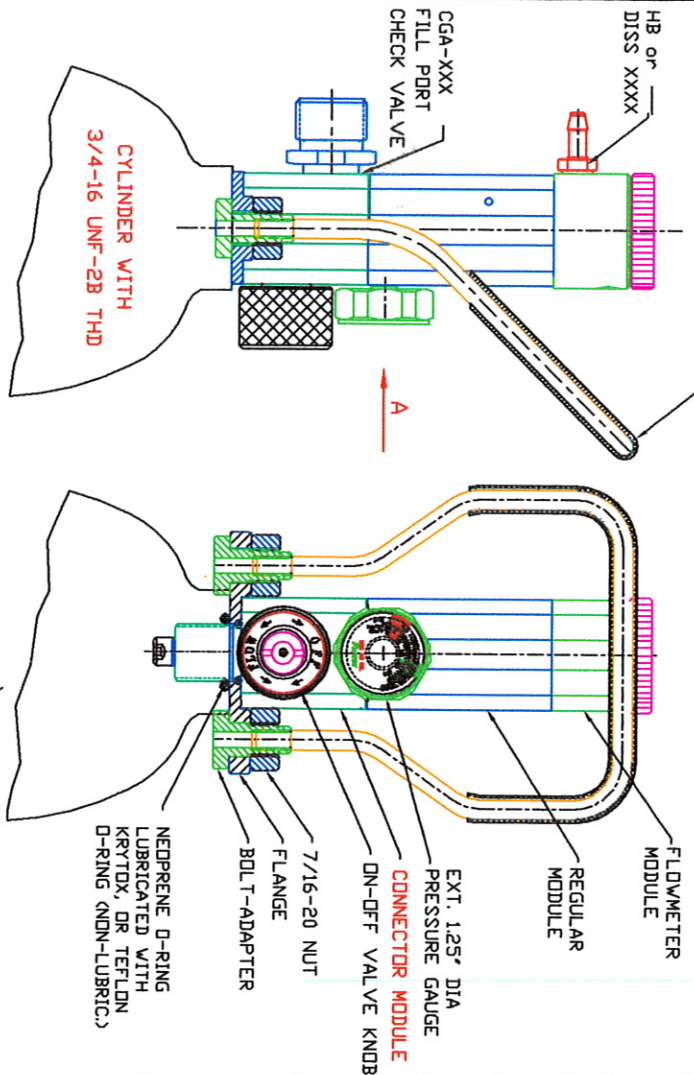


PICTURE #1  
FRONT VIEW  
TO DN-OFF VALVE  
AND EXT. 1.25\"/>

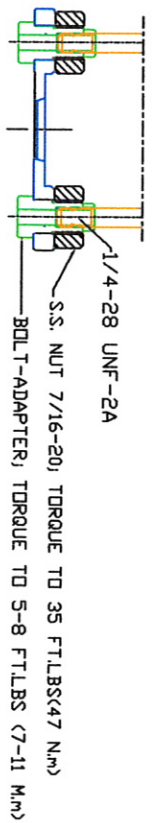

FLOPAC REGULATOR CYLINDER INSTALLATION INSTRUCTIONS

- \* INSPECT FLOPAC CGA-XXX FILL PORT AND BURST DISC FOR CLEANLINESS AND DAMAGE PRIOR TO INSTALLATION.
- \* PERFORM CYLINDER INSPECTION PER CGA ACCEPTED PRACTICES.
- \* SECURE CYLINDER FIRMLY BEFORE VALVING FLOPAC TO ALUMINUM SIZE CG,CD, CE, CJ SIZE CYLINDERS.
- \* FLOPAC'S CONNECTOR BODY VALVING THREADS ARE 3/4-16 STRAIGHT UNF FOR ALUMINUM APPLICATIONS.
- \* REMOVE FLOPAC VALVING THREADS PROTECTIVE COVER.
- \* REMOVE (But do not discard) BOTTOM TEFLON (WHITE) OR NEOPRENE (BLACK) O-RING.
- \* (SEE ENLARGED VIEW OF CONNECTOR WITH DIFFERENT LENGTH THREAD).
- \* INSERT FLOPAC UNIT INTO AND THROUGH HANDLE FLANGE HOLE.
- \* REPLACE O-RING REMOVED IN PRIOR STEP AND THREAD ENTIRE ASSEMBLY UNTIL HAND-TIGHT ENSURING HANDLE-FLANGE WASHER AND FLOPAC BODY ARE SEATED AT A 90-DEGREE PLANE AS DEPICTED IN PICTURE # 1.
- \* USING FLOTEC SUPPLIED SPECIAL 1/2\"/>

ORIGINAL AND REVISED CONNECTOR THREAD CONFIGURATIONS

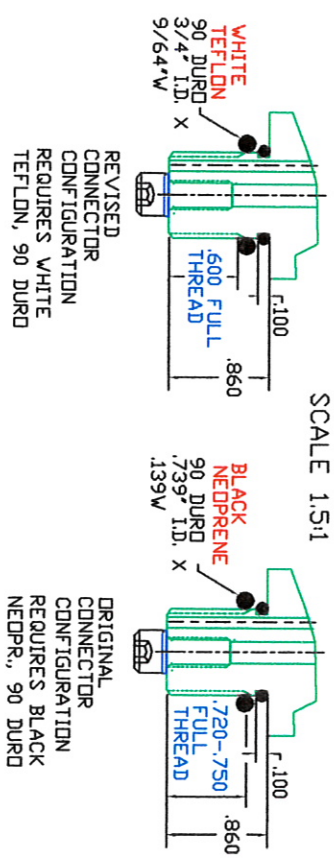

7625 West New York Street  
Indianapolis, IN 46214-4911  
317-273-6879; (Fax) 317-273-6979  
E-mail: flotec@flotec2.com

**CONFIDENTIAL**  
© 2004-07 FLOTEC, Inc.  
INDIANAPOLIS, IN, USA  
ALL RIGHTS RESERVED

CAD # E	CAD FILE # 810-6963-001C	REF# FPB(FP-3); FPC(FP-5)
FLOPAC & 45° BENT HANDLE W/ADAPTER INSTALL INSTRUCTION	SHEET 1 OF 1	DRAWING NUMBER 810-6963-001
		REV. C